

# PRACTI-BABY PACK



## INCLUDES

1 Practi-baby, 1 User manual, 4 lungs, 2 valves and 1 transport bag.

## CERTIFICATE OF WARRANTY



This product has successfully passed the most stringent quality controls before leaving the factory. However, any malfunction due to assembly and / or components faults, please read the warranty and contact with the official distributor of the area.

VICEDO MARTI S.L. warrants this product to be free from defects in material and manufacture at the time of purchase for a period of TWO YEARS from invoice date of the purchase.

If during the period of the warranty, there is a product defect attributable to the manufacturing process, material failure and / or manufacture, the consumer shall inform to the official distributor within 10 business days after learning of the defect, and at the official distributor discretion, the product will either be repaired or replaced.

### CONDITIONS OF WARRANTY:

Details and documentation: A copy of the invoice covering the purchase of your product is required to have warranty coverage. In addition to specifics of the product purchase, the invoice must provide the following information: name and address of the buyer.

Exclusions: This warranty specifically excludes the following:

- Damage and / or defects to the product resulting from misuse, or negligence of the user, including the use of the product for purposes other than its intended use.
- Damage and / or defects caused by the use of external agents such as lime or other noxious chemicals or corrosives.
- Damage and / or defects due to improper maintenance or lack of cleaning.
- Damage and / or defects arising from use of manikin parts not made by the manufacturer of the manikin.

Geographical coverage: The warranty is extended to all countries where this product is sold by a distributor, VICEDO MARTI S.L. or assigned by VICEDO MARTI S.L.

Cancellation of the warranty: Product warranty is cancelled:

- In cases of damage caused by repairs performed by persons other than authorized service agents of VICEDO MARTI S.L., and / or if parts or accessories are used in the manikin other than original parts or accessories manufactured by VICEDO MARTI S.L.
- By changing the data on the invoice, sales receipt or packing slip.



VICEDO MARTI S.L. TEL. +34 965 554 708  
C/ Ávila, 2 - 03440 Ibi - SPAIN

www.practiman.com - info@practiman.com - international@practiman.com




# CPR MANIKIN PRACTI-BABY

## KEY FEATURES AND FUNCTIONS



### ▶ PRACTI-BABY + LIFELIKE INFANT MANIKIN

Practi-Baby's size, weight and movements are suitable for performing correct infant CPR, adapting to training needs.

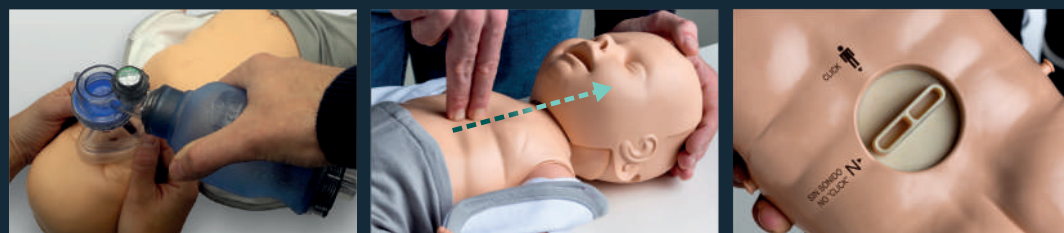

### ▶ PERFORM REALISTIC BREATHING

- ▶ Excellent performance during practice with pediatric resuscitator.
- ▶ The head tilt enables students to learn the correct breathing technique.

### ▶ AUDIBLE FEEDBACK

The selector gives the option of practice sessions with or without clicking.

## OTHER ADVANTAGES OF PRACTI-BABY



### ▶ GUEDEL AIRWAY

- ▶ To practice with a Guedel airway, remove the valve, replace the skin and then measure and insert the airway.

### ▶ CHOKING PRACTICE

### ▶ AED TRAINING

Pads do not leave residue on the skin.

PRACTI-MAN  
CPR TRAINING SOLUTIONS  
by Vicedo Marti

# ASSEMBLING PRACTI-BABY



Remove the chest skin.



Remove the face skin.



Move the lung to below the jaw.



Connect the lung to the valve.



Make sure the valve is correctly attached to the head.



Place the lung on the chest



Fit the face skin, checking that the nose, mouth and upper tab are in the correct position.



Place chest skin and attach to the studs.

# CLEANING AND DISINFECTION RECOMMENDATIONS

To ensure maximum hygiene, the manufacturer recommends thorough cleaning of the Practi-Baby as per instructions below:



The manikin must be cleaned by disinfecting the lips and face, removing the face skin from the head after each training session.



For more thorough disinfection, immerse the face skin in a container of disinfectant, wash carefully and store dry for correct use.



After each use, the Practi-Baby valve must be disinfected and allowed to dry for correct functioning.



The Practi-Baby lung is semi disposable, and can be used to train group of people in a session without needing to change it. If the chest stops rising during the practice session, check that the lung is not moist. If it is, replace it with a new one.

## PRACTI-BABY SPARE PARTS



**LUNG**  
(Ref. SP-001 (PB))



**VALVE**  
(Ref. SP-002 (PB))



**FACE SKIN**  
(Ref. Face-PB)



**CHEST SKIN**  
(Ref. Chest-PB)

## PROTECTION DURING CPR TRAINING SESSIONS



Face shields provide additional protection during training sessions



To keep the manikin perfectly clean, use the sanitising wipes